



Doffing an ear-loop mask for re-use during your shift.*

* Mask should be discarded at the end of your shift.

1. Perform hand hygiene

2. Remove mask

- Remove mask by holding the ear-loops. The front is contaminated, so remove slowly and carefully.

3. After removing face mask, visually inspect for soilage or damage. If soiled, torn or saturated, the mask should be discarded.

4. If the face mask is **NOT** visibly soiled, torn or saturated, **carefully store on a clean paper towel, exterior (colored) side down.** (Exterior of ear-loop mask should be placed directly on the paper towel.)

5. Perform hand hygiene

